ITALIANO

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PROCARE®

SPORT Ankle Splint/ Lace-Up Ankle Splint

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13-1382-0-00000 REV F

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The user should be able to:

- Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS:

This device is designed to:

- Promote healing and reduce further damage after injury.
- Provide compression and support to the injured area.
- Helping to stabilize the ankle to prevent any unwanted movement that could aggravate injury.
- Enhancing comfort during activity.
- Also for patients with Acute ankle sprains, Chronic ankle sprains and Prophylactic use.pain.

PERFORMANCE CHARACTERISTICS:

Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

WARNINGS AND PRECAUTIONS: Do not use if allergic to neoprene. Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene by products. A small percentage of people develop an allergic condition to neoprene. Should rash or skin condition develop, discontinue wearing the product immediately. If the condition persists, consult your physician. Do not use over open wounds. If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately. Do not use this device if it was damaged and/or package has been opened.

Note: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INFORMATION:

1. While seated at the edge of a chair, place cotton sock over the foot and ankle. Nylon stockings may not be worn with this brace. (Fig. 1)
2. Before application, loosen or remove laces. Slip the foot through the support. (Fig. 2)
3. Lace up the support as you would a high-top shoe. Laces should be snug, but not restrict circulation. (Fig. 3)

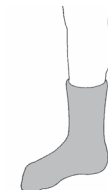
Figure 1

Figure 2

Figure 3

CLEANING INSTRUCTIONS:

To clean, hand wash with mild soap in lukewarm water. Rinse thoroughly to remove all residual soap. AIR DRY.

Note: Residual soap may cause skin irritation and deteriorate the material.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2020 DJO, LLC