ITALIANO

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13-1383-0-00000 REV D - 2020-12-16

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS

This device is designed to provide support for the ankle following injuries such as ankle sprains and chronic ankle instabilities. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

CONTRAINDICATIONS: N/A

APPLICATION INFORMATION:

A) For application, first unfasten forefoot and ankle straps. While seated, step into brace, secure forefoot strap (#1), then wrap and secure lower ankle strap (#2). (Fig. 1)

Figure 1

B) Pull up on heel counter (#3) until snug and secure. (Fig. 2)

Figure 2

C) Wrap and secure the upper ankle strap (#4). Readjust lower ankle strap if necessary. (Fig. 3)

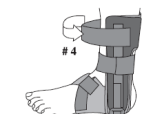
Figure 3

CLEANING INSTRUCTIONS: Hand wash in cold water with mild soap, rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause irritation or deteriorate material.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

WARNINGS AND PRECAUTIONS:

Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene byproducts.

- 1) Do not use over open wounds.
- 2) If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- 3) Do not use this device if it was damaged and/or package has been opened.
- 4) If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

FOR SINGLE PATIENT USE ONLY.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2020 DJO, LLC