

DonJoy's Arch Rival[™] is a patented pre-fabricated orthotic designed to treat Subtle Cavus Foot (higher arches and heel varus) by promoting natural foot function, improving stability, adding shock absorption and alleviating pain.

Diagnosis:

Subtle Cavus Foot (SCF) is characterized by higher arches and heel varus – conditions that may be difficult to identify during a physical examination. Below is a useful method for diagnosing SCF:

Peek-A-Boo Heel Heel pad easily seen

Heel pad easily seen medially from the front with the patient standing with feet straight ahead.



60-Day Satisfaction Guarantee

Congratulations on your purchase of the DonJoy Arch Rival™! Please be assured that it is made of the finest materials and most advanced technologies available. If you are not fully satisfied, simply contact Customer Service at (800) 336-6569 within 60 days of your purchase for a refund. Any alteration beyond the suggested fitting instructions will result in termination of this guarantee.

Application Instructions:

Before use, please read these instructions completely and carefully. Correct application is vital to the proper functioning of the orthotic.

ndications

• Subtle Cavus Foot (higher arches and heel varus).

Fitting Instructions:

- Approximate sizing is determined by shoe size.
- Appropriate fit allows your 1st metatarsal head to sit in the recessed area with your heel comfortably placed in the back of the orthotic. Note: You may need to size up or down for optimal fit.
- Using the insole from your existing shoe as a template, trim the Arch Rival with scissors to match the size.





• Place securely into shoe, ensuring the Arch Rival is seated properly and toe area is not folded over.

Break-In Instructions:

- Allow 1-3 weeks for proper break-in.
- For first time use, do not exceed 1-2 hours.
- If there are no side effects, increase use by 1-2 hours per day until full time use.

Warranty Informatio

The Arch Rival™ comes with a 3 month warranty against defects on soft goods associated with the orthotic. Any alteration beyond the suggested fitting instructions will result in termination of the warranty.

Use and Car

- Use a soft brush with anti-bacterial soap and warm water
- Let insole air dry completely before reinserting into shoe
- Store in a cool, dry place
- Do not machine wash or dry

Warning: If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

Caution: Federal Law (U.S.A.) Restrict this device to sale by or on the order of a licensed health care professional.






What is Arch Rival™?

DonJoy's Arch Rival™ is a patented pre-fabricated orthotic designed to treat Subtle Cavus Foot (higher arches and heel varus) by promoting natural foot function, improving stability, adding shock absorption and alleviating pain.

How does Arch Rival[™] work?

A patented recess under the first metatarsal head accommodates a plantarflexed first ray and allows hindfoot eversion, unlocking the subtalar joint. The elevated heel reduces shock and accommodates for tight calf musculature.

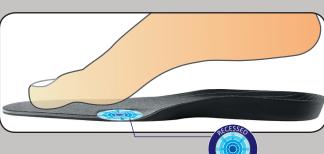

X-static Technology

The Arch Rival orthotic is designed with silver-coated fibers woven into the surface layer of fabric to discourage bacterial and fungal growth, blisters and odor.



Thermodynamic

Silver is the world's most effective thermally conductive element. Silver fibers in the Arch Rival work to evenly distribute heat throughout the orthosis, while also decreasing friction, hotspots and blisters.

• Anti-Microbial

Tests have shown that X-Static technology actively eliminates 99% of bacteria and fungi growth on the orthotic.

Anti-Odor

By binding with ammonia and denatured proteins, X-Static technology naturally eliminates odor-causing bacteria.

Arch Rival[™] **Sizing**

Approximate sizing is determined by shoe size. Due to the unique design of the Arch Rival, please refer to the fitting guide for precise instructions. Arch Rival fits in most athletic and walking shoes.

	Α	В	С	D	E	F	G
WOMEN	4.5-6	6.5-8	8.5-10	10.5-12	12.5-14	14.5+	
MEN		5.5-7	7.5-9	9.5-11	11.5-13	13.5-15	15.5-17



DJO, LLC 1430 Decision Street Vista, CA 92081-8553 U.S.A. t 800.793.6065 f 800.457.4221

© 2010 DJO, LLC US Patent #: 6604301 13-9238_B







