PATIENT INSTRUCTIONS

WRIST BRACE

GIVE TO PATIENT

WRIST BRACE

ORTESIS DE MUÑECA

EXOS® HANDELGELENKHOESE OFFEN

TUTORE PER POLSO

ATTELLE DE POIGNET

WRISTBRACE

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WB_10128 REV E 2021-03-03

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DJO, LLC
1430 Decision Street
Vista, CA 92081-8553 • USA

ENGLISH

Before using the device, please read the entire instruction manual. Correct application and care is vital to the proper functioning of the device.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE / INDICATIONS

The Wrist Brace (WB) provides stabilization to the wrist joint.

CONTRAINDICATIONS

Do not use if you are allergic to any of the materials contained within this product.

⚠ WARNINGS AND PRECAUTIONS

Instruction for wearing, opening, closing or removing the brace should be provided by a physician.

- Consult your physician if unusual swelling, increasing discomfort, pain, numbness or discoloration appears.
- If brace does not fit properly, or is irritating, it may be reformed and adjusted by a healthcare professional.
- If brace breaks or cracks, see your healthcare professional for a replacement.
- Wearing the brace too tight may result in skin irritation, odor, or other skin issues.
- If the brace is applied with a Locking Ring or the patient is instructed not to loosen or remove the brace, the patient must not get brace wet.
- The brace may soften and lose its desired shape when exposed to heat over 130°F (55°C).
- Do not expose brace to moisturizers, chemicals, or solvents, as they may affect durability.

OPERATING LACING SYSTEM

- To Loosen:
 - Lift hook-and-loop fastener tab.
 - Pull open brace. (Figure 1)

To Close and Tighten:

- Pull hook-and-loop fastener tab to cinch lace. (Figure 2)
- Wrap excess lace around brace.
- Secure hook-and-loop fastener tab to brace.

3 KEYS FOR CARE AND COMFORT

1. Not Too Tight - Wear it Right and leave some "WIGGLE ROOM!" (Figure 3)

- Exos® Braces are meant to be worn under light pressure and not tight to the skin. A little wiggle room inside an Exos® Brace promotes air circulation and helps maintain dry healthy skin.
- An over-tightened brace creates "shear" or pressure on the skin during motion, which can lead to irritation, redness, and blisters.
- Wearing the brace too loose will reduce support and may compromise healing.

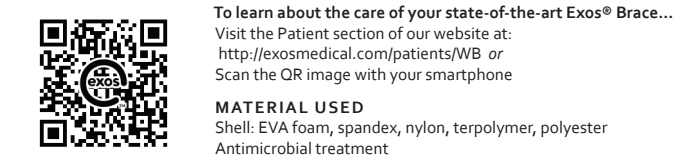
2. If the brace and skin get wet, they "CANT STAY WET"

- The Brace may be washed or worn while swimming or bathing, only if the physician allows the patient to loosen or remove the brace. If not, the patient cannot get the brace and skin wet.
- It is imperative that the inside surface of the brace and the patient's skin underneath are completely dry after swimming or bathing.
- See below for cleaning and drying instructions.

3. Keep your "SKIN AND BRACE CLEAN AND DRY"

- Exos® braces are treated with antimicrobial to help eliminate odor causing bacteria. However, when possible we recommend cleaning your brace.
- If your physician has given permission to loosen or remove the brace, wash the brace (inside and out) and your skin with antibacterial soap, rinse thoroughly, and completely dry the skin and brace (inside and out) with a hair dryer.
- A clean and dry environment inside the brace will help maintain healthy skin and reduce odor and skin issues.

- Loosen or remove the brace as instructed by physician.
- Wash inside and outside of brace and skin with antibacterial soap and water (86°F / 30°C). (Figure 4)
- Rinse thoroughly with water.
- Use a hair dryer set on high volume and cool setting to thoroughly dry the interior of the muñequera y la piel. (Figure 5)
- Retighten the brace.

LIMITED PRODUCT WARRANTY
DJO, LLC will repair and replace all or part of the unit and its accessories for material or workmanship defect for a period of 8 weeks from the date of sale.

To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

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