ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE/INDICATIONS: Mild wrist sprains and strains.

CONTRAINDICATIONS: None

WARNINGS AND PRECAUTIONS:

- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene by-products.
- If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.
- Do not use this device if it was damaged and/or packaging has been opened.

APPLICATION INFORMATION:

- Position sleeve on thumb with ends pointing towards body.
- Wrap the longer tab around the wrist, over the shorter tab and secure.

CLEANING INSTRUCTIONS: Hand wash in cold water using mild soap, rinse thoroughly. AIR DRY. **NOTE:** If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

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©2021 DJO, LLC
MDSS GmbH
U.S.A.